fr

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en

**INSTRUCTIONS FOR USE**

**Please read instructions carefully prior to use.**

**Sterile absorbable collagens for use as hemostatic agent after tooth extractions and oral surgery**

**APPLICATION FORM**

The sterile absorbable SeptoCone is used as hemostatic agents after tooth extractions and oral surgery. The application of SeptoCone into the defect of cavity supports the stabilization of the produced blood clot whilst helping to control bleeding at the application site.

**COMPOSITION**

1 SeptoCone cone contains 22.4 mg of native collagen fibrils of equine origin.

**INTENDED USE**

The sterile absorbable SeptoCone is used as hemostatic agents after tooth extractions and oral surgery. SeptoCone can be combined with fibrin glue in patients with an increased risk of bleeding.

**COMPOSITION**

1 SeptoCone cone contains 22.4 mg of native collagen fibrils of equine origin.

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**Medical treatment of severe bleeding disorders is mandatory prior to the beginning of the procedure.**

Unless otherwise prescribed, SeptoCone can be applied directly into the extraction alveola. The SeptoCone moulds itself into the site of the extraction or excision wound, stop bleeding, and will be absorbed.

**CONTRAINDICATIONS**

- SeptoCone must not be used:
  - on patients with a known hypersensitivity to products of equine origin,
  - in infected areas,
  - during pregnancy and lactation.

**WARNINGS**

Chlorine-releasing antiseptic agents, tannins and caustics should not be used in combination with collagen.

SeptoCone is not intended to be sutured through; doing so may compromise the performance of the SeptoCone.

**SIDE EFFECTS**

Due to the xenogenic origin of the collagen, there is a possible risk of intolerance reaction, despite the confirmed low antigenicity.

**FURTHER NOTES**

Due to the affinity of collagen to bleeding surfaces, dry instruments and gloves must be used when applying SeptoCone. The product is intended for single use only. SeptoCone must not be reused, there is a risk of infection for the patient and/or user and the functionality of the product for the patient and /or user cannot be guaranteed. Remaining material, left over due to cutting for example, must be disposed of according to local procedures and guidelines.

**NOTES FOR POST-OPERATIVE CARE FOLLOWING DENTAL SURGERY**

**To be avoided:**

- Rinse your mouth out on the day of surgery (may dislodge clot).
- Drink alcohol or hot drinks on the day of surgery (may increase blood pressure or induce hyperaemia).
- Smoke for at least 24 hours (causes vasoconstriction).
- Carry out physical activity on the day of surgery (increasing blood pressure).

**Recommend:**

- Sip lukewarm drinks carefully (helps avoid clot removal by washing out socket).

- Bite on gauze or clean, damp handkerchief for 10 minutes if area bleeds, preferably sitting upright in a chair.
- Rinse mouth gently with hot salt mouthwash or Chlorhexidine (0,2 %) mouthwash after 24 hours. Continue regular mouthwash use over the following week, especially after meals.
- Clean teeth as normal, avoiding area of surgery for 24 hours.

**STERILITY**

SeptoCone may not be resterilised and opened single packs must be discarded. Do not re-use SeptoCone in whole or in part, as it may compromise sterility and /or the performance of SeptoCone.

**STORAGE**

Store between 0°C -25°C.
Store protected from moisture and direct sunlight.

Do not refrigerate.
Store out of the sight and reach of children.
PRODUCT CONFORMS TO THE ESSENTIAL REQUIREMENTS OF THE COUNCIL DIRECTIVE 93/42/EEC CONCERNING MEDICAL DEVICES.

**DESCRIPTION OF THE SYMBOLS USED ON THE PACKAGE**

| REF                                                                                   | REFERENCE NUMBER                                          |
|---------------------------------------------------------------------------------------|-----------------------------------------------------------|
| LOT                                                                                   | BATCH NUMBER                                              |
|    | USE BY YEAR – MONTH – DAY                                 |
|    | CONSULT INSTRUCTIONS FOR USE                              |
|    | DO NOT RE-USE                                             |
|    | DO NOT RSTERILIZE                                         |
|    | DO NOT USE IF PACKAGE IS DAMAGED                          |
|    | STERILIZED                                                |
|    | STERILIZED USING ETHYLENE OXIDE                           |
|    | 0 °C – 25 °C                                              |
|    | PROTECT FROM DIRECT SUNLIGHT                              |
|   | HIBC CODE                                                 |
|  | KEEP DRY                                                  |
|  | MANUFACTURER                                              |
|  | CE MARKING AND IDENTIFICATION NUMBER OF THE NOTIFIED BODY |

For dental professional use only
GA 0089, Status 2021-03
REF : DK1010SD

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TRANSLATION • INTERPRETING • SUBTITLING

**To whom it may concern**

I, Jane Crossley, MA, Project Manager for Andiamo!, Devonshire House, Devonshire Avenue, Leeds LS8 1AY, England, hereby certify that the annexed translations in the languages BULGARIAN, CZECH, DANISH, DUTCH, ESTONIAN, GERMAN, GREEK, ITALIAN, HUNGARIAN, LATVIAN, LITHUANIAN, NORWEGIAN, POLISH, PORTUGUESE, ROMANIAN, SLOVAK, SLOVENE, SPANISH, and SWEDISH which is based on the translation originally delivered on 12<sup>th</sup> May 2021 and includes subsequent changes to Dutch, Estonian, German, Lithuanian, Polish, Slovene, and Slovak requested by Advanced Medical Solutions (Resorba) and of the following documents are, to the best of my professional knowledge and belief, true and faithful renderings of the ENGLISH original:

- GA 0089\_IFU\_SeptoCone\_21spr\_2021-03

And from ENGLISH into BULGARIAN, CZECH, DANISH, DUTCH, ESTONIAN, GERMAN, GREEK, ITALIAN, HUNGARIAN, LATVIAN, LITHUANIAN, NORWEGIAN, POLISH, PORTUGUESE, ROMANIAN, SLOVAK, SLOVENE, SPANISH, and SWEDISH

- GA 0089-IFU\_SeptoCone\_Titel+Rck\_2021-04

A handwritten signature in blue ink, appearing to read "Jane Crossley".

Jane Crossley, MA

Date: 12<sup>th</sup> July 2021

Andiamo! HQ: Devonshire House, Devonshire Avenue, Leeds LS8 1AY

Tel: 08450 345677 | E-mail: [info@andiamo.co.uk](mailto:info@andiamo.co.uk) | Website: [www.andiamo.co.uk](http://www.andiamo.co.uk)

Andiamo! Language Services Limited trading as Andiamo! Registered address: 35 Rodney Road, Cheltenham, GL50 1HX. Registered No 4123249