I	R41	Rischio di gravi lesioni oculari.	D	R41	Gefahr emster Augenschäden.	Xi  IRRITANTE IRRITANT REIZEND
	S26	In caso di contatto con gli occhi, lavare immediatamente e abbondantemente con acqua e consultare un medico.		S26	Bei Berührung mit den Augen sofort gründlich mit Wasser abspülen und Arzt konsultieren.	
	S39	Proteggere gli occhi/la faccia.		S39	Schutzbrille/Gesichtsschutz tragen.	
GB	R41	Risk of serious damage to eyes.	E	R41	Riesgo de lesiones oculares graves.	Prodotto da: Manufactured by: Produit par: Hergestellt von: Fabricado por: MAGNOLIA s.r.l. Via Natta, 6/A 43100 PARMA ITALY Tel. +39 0521607604 fax +39 052139967 Email: magnoliasrl@virgilio.it
	S26	In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.		S26	En caso de contacto con los ojos, lávense inmediata y abundantemente con agua acúdase a un médico.	
	S39	Wear eye/face protection.		S39	Úsese protección para los ojos/la cara.	
F	R41	Risque de lésion oculaires graves.	P	R41	Risco de lesões oculares graves.	
	S26	En cas de contact avec les yeux, laver immédiatement et abondamment avec de l'eau et consulter un spécialiste.		S26	Em caso de contacto com os olhos, lavar imediata e abundantemente com água e consultar um especialista.	
	S39	Porter un appareil de protection des yeux/du visage.		S39	Usar um equipamento protector para os olhos/face.	

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SANITIZING ANTI-FOAMING FOR DENTAL ASPIRATORS

Blood, mucus and all kind of sanitizing substances, in presence of the turbulence created by aspiration, build-up a lot of foam, which can cause frequent and sudden unwanted stop of aspiration. In the morning, before starting to work, or in the evening, after the usual disinfecting washing of the aspiration system, we suggest to put one or two tablets in the filter of the tip support or in the one of the canister. Each tablet is wrapped in a water-soluble protective film, ensuring preservation and safe handling. The protection film shall not be removed, as it will dissolve with water. Aspirating a small quantity of water through a tip, after placing the tablet inside the tip support filter or before starting the aspiration, is enough to obtain an immediate anti-foaming efficient effect. If the tablet has to be placed in a small space, remove the protective film (use of gloves is recommended) and break it in two parts by pressing along the pre-cut notch. The tablet will melt slowly as liquid passes by, and will liberate the antifoaming agents all the working day long. Any undissolved debris of tablet can be removed from filters (better once the suction system daily disinfection has been carried out) by using running hot water. If the filter is very small and a split tables is used, its solubility and antifoaming efficiency are increased, whilst its life is slightly reduced. The antifoaming sanitizing tablet contains also a bactericidal active ingredient to avoid the growth of bacteria, which is a possible source of smells.

Certified Quality System UNI EN ISO 9001:2000.

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