

**RELIANCE PLASTIC APPLIANCE
CONDITIONER****(Ref# PL)**

INDICATIONS FOR USE: The **Reliance Plastic Appliance Conditioner** is intended for use as a conditioner of plastic brackets or acrylic appliances in preparation for bonding with any adhesive, sealant or cement. It can also be used to condition a composite restoration or acrylic jacket prior to bonding and to rebond a bracket impregnated with adhesive.

INSTRUCTIONS FOR USE:**BONDING A PLASTIC BRACKET OR ACRYLIC APPLIANCE:**

With a brush or sponge pellet, apply a thin uniform coat of **Reliance Plastic Appliance Conditioner** to the plastic surface to be bonded. Wait at least 60 seconds before applying the bonding paste.

BONDING TO A COMPOSITE RESTORATION:

With a fine diamond, lightly abrade the composite to remove the shine. Clean area with an air syringe. Using a sponge pellet or brush, apply a thin uniform coat of **Reliance Plastic Appliance Conditioner** and allow to dry. Apply a thin coat of bonding resin to the conditioned composite surface and the paste to the bracket. Place bracket on tooth and position. If Rely•a•Bond® is the adhesive of choice, apply the Rely•a•Bond® Primer to the conditioned composite and the bracket base. Apply the paste to the bracket base, place the bracket on the tooth and position.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



TEMPERATURE LIMITATION: Store at room temperature (not to exceed 86 degrees F) or can be refrigerated (36-46 degrees F). Refrigeration will prolong the shelf life of the **Reliance Plastic Appliance Conditioner**. Avoid excessive heat.



Flammable

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.




Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com



Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

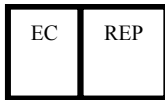

EMERGO EUROPE
Molenstraat 15
2513 BH, The
Hague
The Netherlands

Scan for more information